


Domus Auto

Automatic pressure redistribution system for optimizing care process

Domus Auto dynamic mattress system automatically calibrates in accordance with the patient's weight and position in the bed, eliminating the risk of injury due to the caregiver incorrectly entering the patient's weight.


Additionally, the mattress provides extra pressure support during sitting to prevent bottoming out, and the max firm mode makes it ideal for nursing tasks.

Domus Auto's effective pressure relief and safety features, including an alarm system and automatic panel lock, further contribute to optimized care delivery. Additionally, the mattress operates quietly, minimizing disturbance and enhancing overall satisfaction.¹

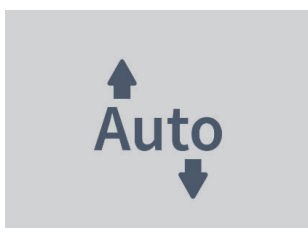


Enhance Care Process

Automatically adjusts to the most optimal pressure for the patient, ensuring both clinical effectiveness and cost efficiency.

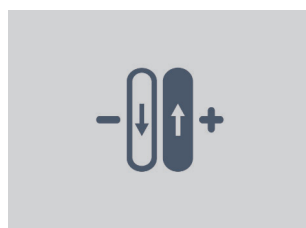

Customized Firmness for Comfort

Tuning function improve patient comfort by manually micro-adjust mattress firmness.

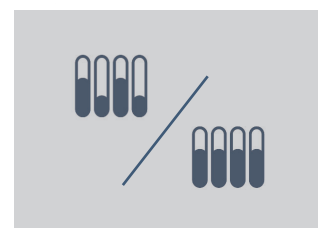

Therapy Modes to Meet Diverse Needs

Provide Continuous Low Pressure (CLP) mode for enhanced patient immersion, Alternate mode for allowing pressure redistribution, and Max Firm mode for nursing procedures.

¹. Wellell internal Non-interventional study report

Domus Auto


Enhancing care processes with implementing early pressure injury prevention



CPR Knob
Quickly deflate the mattress for emergencies.



Special Design of Air Cells
Cell-on-cell design for continuous pressure support, even during power failures.

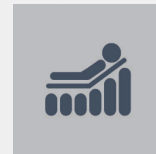


Prevent Bottoming Out while Sitting
Seat inflate mode manually manages the inflation of the air cells to improve stability and comfort during sitting.



Storage Strap
Side buckle under the base cover can be used to store rolled mattress very easily.



Quick Connector with Cap
Mattress can be transformed from a powered support surface to a sealed static air mattress for transportation need.

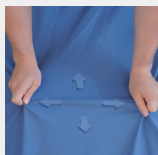

Non-stretch Top Cover with Cotton Fabric
Offering low shear, low friction, vapour permeability, and moisture protection, coupled with optimal immersion and envelopment.



Water Resistance




Flame Resistance




Biocompatibility



Vapour Permeable

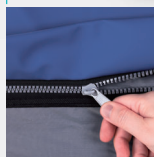

Decreases the Risks of Infection
The top cover is made with welded seams (high frequency induction hardening [®]) to avoid fluid ingress.

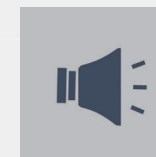

Prioritize Patient Safety
Visual and audible signals are used to alert caregivers in case of any failure.

Specifications

Model	Domus Auto				
Pump	Dimension	29 x 18.5 x 12.6 cm (11.4 x 7.3 x 5 in)			
	Weight	2.3 kg (5.1 lb)			
	Case Material	Fire Retardant ABS			
	Supply voltage	100-120 V / 60 Hz			
	Therapy mode	Max firm / Alternate / Static / Seat inflate			
	Noise Level	< 30dB			
Mattress	Dimension	Cells	Length	Width	Height
		20	200cm (78.7 in)	90cm (35.4 in)	20.3cm (8 in)
	Type	8" / Replacement			
	Weight	6.9 kg (15.2 lb)			
	Top cover material	Nylon / PU with Cotton Fabric			
	Cell material	Nylon / TPU			
	Maximum patient weight	200 Kg (450 lb)			
	HPCPS Code	E0277			
Ordering Information (For North America Only)					
System	With Quilted Cover	PD03501			
Mattress	With Quilted Cover	PM09401			
Pump		PD03001			

Wellell America Corp.

927 Mariner Street,
Brea, CA 92821, USA
T +1 714 671 3818
F +1 714 494 8173
info.usa@wellell.com
www.wellell.com
Print-2024/07 All rights reserved