# Sedens 400

User's Manual

Care for a Healthy Life

## 1. Intended Use

This Apex Hybrid Seat Cushion is an adjustable, air-and-foam filled, cellular-design wheelchair support surface that intended to provide continuity of pressure care from bed to chair. The cushion is compatible with all pumps that use 1-in-2 Alternating Apex quick connectors (except Auto systems) to have alternating functions. There is no weight limit, yet the cushion must be properly sized to the individual. It also provides following purposes: to help and reduce the incidence of pressure ulcers while optimizing patient comfort.

### NOTE, CAUTION AND WARNING STATEMENTS

- NOTE:** Indicates some tips.  
**CAUTION:** Indicates correct operating or maintenance procedures in order to prevent damage to or destruction of the equipment or other property  
**WARNING:** Calls attention to a potential danger that requires correct procedures or practices in order to prevent personal injury.


### SYMBOLS

	Refer to instruction manual/ booklet		Humidity Limitation
	Do Not Dry Clean		Do Not Iron
	Tumble Dry, Normal, Low Heat		Machine wash, regular/ normal, 90 degree C (194 degrees F)
	Do Not Bleach		Temperature Limitation
	Manufacturer		Authorised representative in the European community

## 2. Cushion Description

- Stretch Cover
- Zipper
- Air cells with foam inside
- Quick Connector (to Apex pump)
- Cap with Pressure Adjustment Valve


**NOTE:** Please refer to the pump manual for the respective pump description.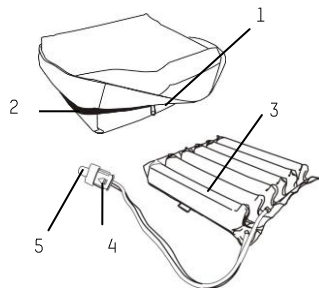

## 3. Installation & Adjustment


### 3.1 Alternating Hybrid Cushion Use- With Apex Pump

**NOTE:** The Sedens 400 Hybrid Cushion is compatible with all pumps that use 1-in-2 Apex Quick Connectors, except Auto systems.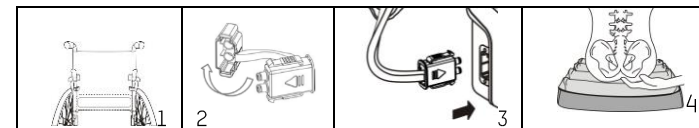

- Place Sedens 400 hybrid cushion in wheelchair seat so the user will be properly supported and immersed when in their normal sitting position.
- Uncapped the quick connector and connect to a compatible Apex pump. A "click" sound will be heard when the connection is completed.
- Install the pump according to the pump manual.
- Have the user sit on the cushion. Check to make sure the cushion is properly sized to the user and wheelchair.
- Adjust an appropriate pressure setting to ensure the patient's comfort without bottoming out.
- Keep hand under user's lowest bony prominence and make sure there is between 1/2 in. (1.3cm) to 1 inch (2.5 cm) of cushion between the user's lowest bony prominence and the seating surface. If not enough space in between, please adjust the pump pressure to higher level.

### 3.2 Temporary Static Cushion Use (without pump)

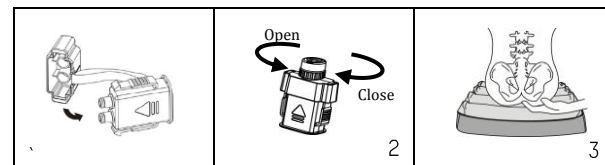- Capped on the quick connector and open air valve by turning valve **counterclockwise**.
- WARNING:** Make sure cushion is unweighted to inflate. (No patient should be sitting on cushion at this time)
- When cushion is fully inflated, close air valve by turning valve **clockwise**.
- Have the user sit on the cushion. Check to make sure the cushion is properly sized to the user and wheelchair.
- Turn the valve **counterclockwise** to release air, while keeping hand under user's lowest bony prominence. Release air until there is between 1/2 in. (1.3cm) to 1 inch (2.5 cm) of cushion between the user's lowest bony prominence and the seating surface.
- Turn the valve **clockwise** to close.

## 4. Cleaning

Cover Material : Stretch	
--------------------------	--

Wipe-down the mattress unit with a damp cloth pre-soaked with warm water containing a mild detergent, or chlorine bleach followed by an approved intermediate level disinfectant. Top cover can also be completely removed for laundry with indicated temperature; however, it is recommended that the user still check with local policy to determine the time/ temperature ratio required achieving thermal disinfection. After cleaning, please avoid direct sunlight or dust areas. All parts should be air dried thoroughly before use.

## 5. Technical Specifications

Cushion	Specification
Dimension (L x W x H)	44 X 41 X 9 cm or 17.3" x 16.1" x 3.5"
Weight	1.1 kg or 2.4lb
Max. Weight Capacity	There is no weight limit, yet the cushion must be properly sized to the individual

### NOTE:

- Pump specification/EMC information please refer to the pump manual.
- Consult the distributor for other technical documents.
- Please follow national requirements to dispose of the unit properly.
- The manufacture reserves the right to change product specification without prior notice.

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Print-2018/All rights reserved
**Apex Medical Corp.**
Apex Medical (Kunshan) Corp.